## BL-F(D) Operating Instruction 사용설명서

### 제품 보증서 QUALITY GUARANTEE

PRODUCT (제품)	
Model No. (모델명)	
Serial No. (일련번호)	
Purchase date (구입일자)	
CUSTOMER (고객)	
Company (회사명)	
Address (주소)	
Tel. (전화)	
E-Mail (이메일)	

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#### English

- The legal warranty period for the perfect functioning of purchased goods is 12 months from date of purchase.
- For the manufacture of this product, only first class, proved material was used with the utmost care. Anyway, should you recognize a failure, based on material errors or incorrect assembly can be attributed. We will remove the source of this error immediately and free of charge. You have to enforce your claims in written from during this warranty period.
- Defects due to improper handling, installation error, excessive usage, misapplication or damage resulting from inexpert repairs or modifications by either your-self or by any third party are not covered by the warranty.
- Also defects, which are based on normal abrasion are excluded from warranty.
- In particular, we do not assume, as far as permitted by law, no liability for consequential damages who are caused by the device.
- Claims of other species, such as rescission, mitigation or compensation are excluded by the warranty.
- The warranty is only valid to the company of the first purchaser.
- Be sure to keep your sales receipt and the completed warranty slip in a safe place ; this will facilitate any future warranty claims.
- Shipment of defect devices, has to occur exempt free from postage for us.

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Ver. 2019-04

### IMPORTANT INFORMATION

Read this instruction before starting up the machine carefully to avoid incorrect operation or other damage. If other language version is required, please request our agent in your area. Duplication and distribution of this instructions require MICRO-NX's prior consent.

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Manufacture has right to do modification and improvement for the product's quality.

But It does not imply any right to retrofitting existing units.

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MICRO-NX is not responsible for damage caused by below things.

- Damage caused by nonfulfillment of this instruction.
- External cause like improper installation etc.
- Improper use.
- Repair performed by unqualified person.

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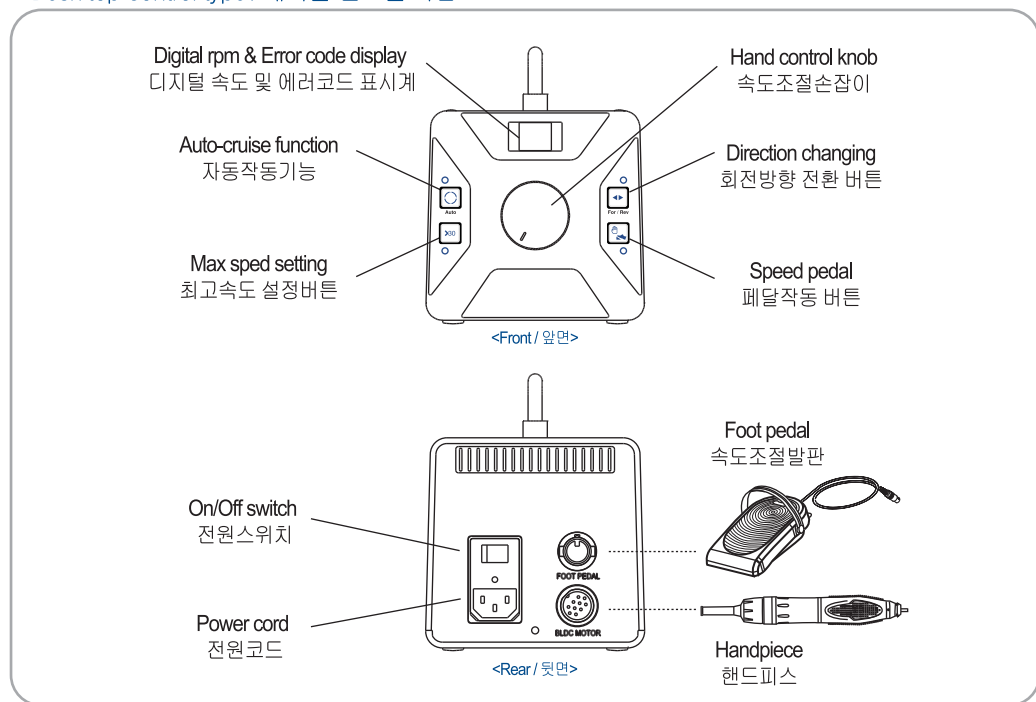
MICRO-NX recommend using only original spare parts supplied by us for operation and repair.

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Desk top control type / 테이블 컨트롤 타입

#### How to operate the control box

Please check the speed knob is positioned at 'zero' and turn on the power switch on the back of the control unit. Initial function configuration when the power is "ON".

- Hand/Foot mode : Hand mode (LED is "Lighting")
- FWD/REV mode : FWD mode (LED is "Lighting")

#### How to use 'Foot speed control'

1. Please set up 'Foot mode' by Hand/Foot mode button. (LED is off when 'Foot Mode')
2. Please set up the maximum speed by speed knob on the front of the machine.
3. When you step on the foot pedal, the motor rotates up to maximum speed you set.

#### How to use 'Auto-cruise' function.

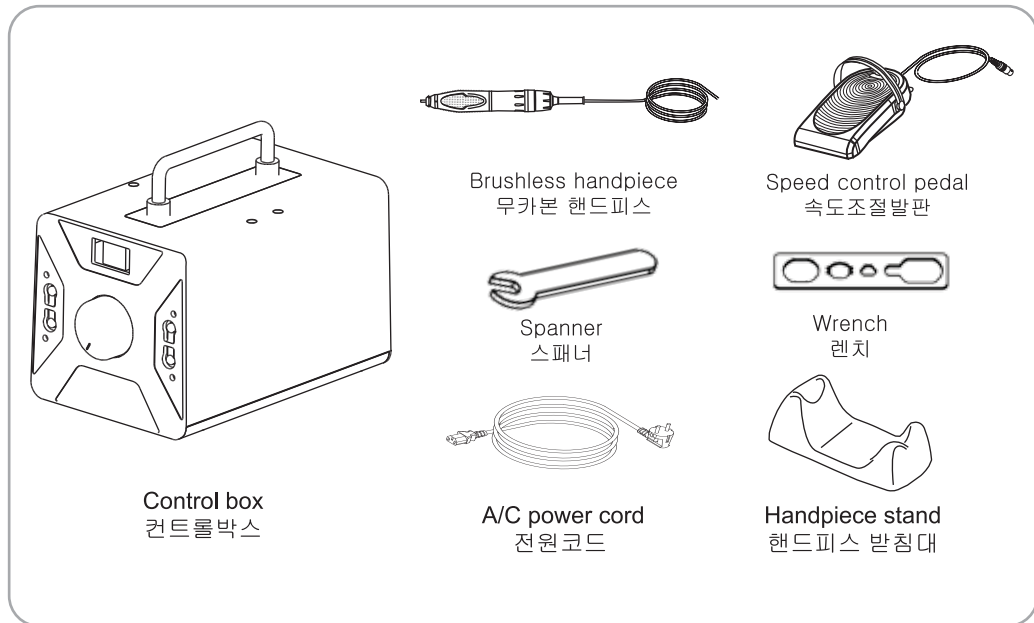
1. When you use foot speed pedal, push 'Auto cruise' button.
2. Step on the foot pedal and stay for 2~3 seconds at the speed that you want , after beep sound, the motor rotates automatically at that speed without touching the pedal.

#### How to use 'Maximum speed setting'


1. You can set up machine's maximum speed 30,000 rpm or 50,000 rpm by the button.

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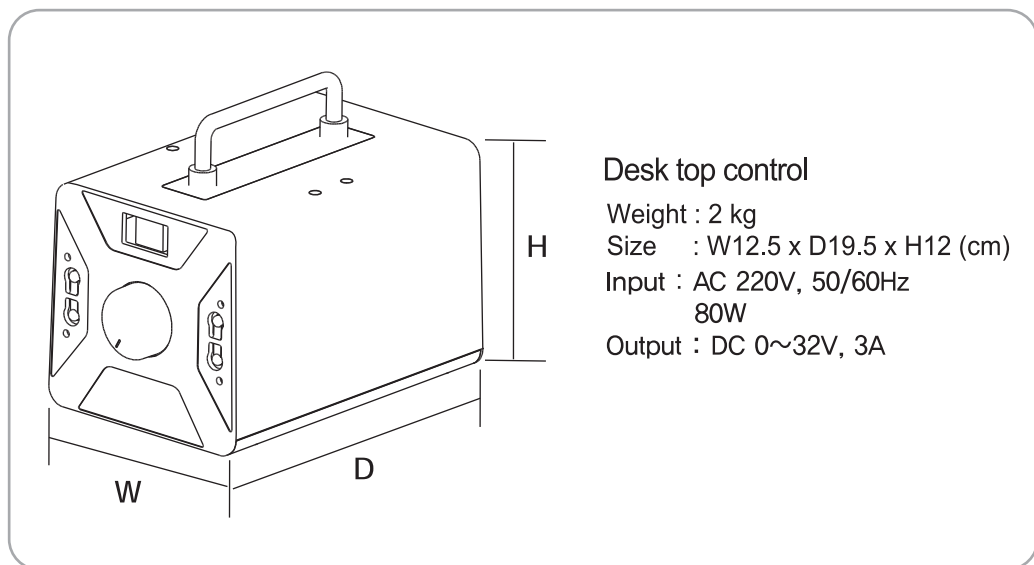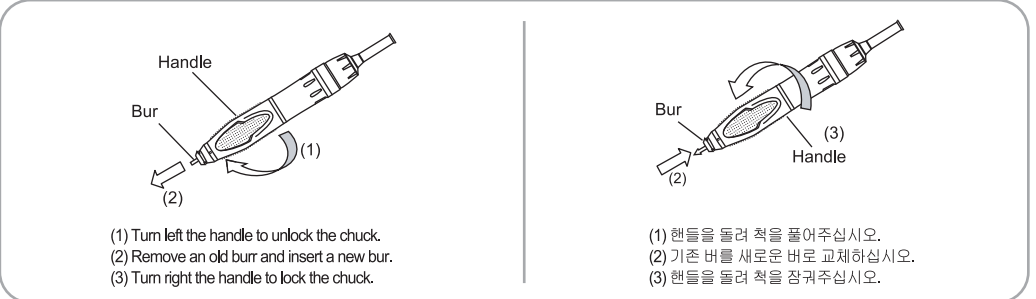
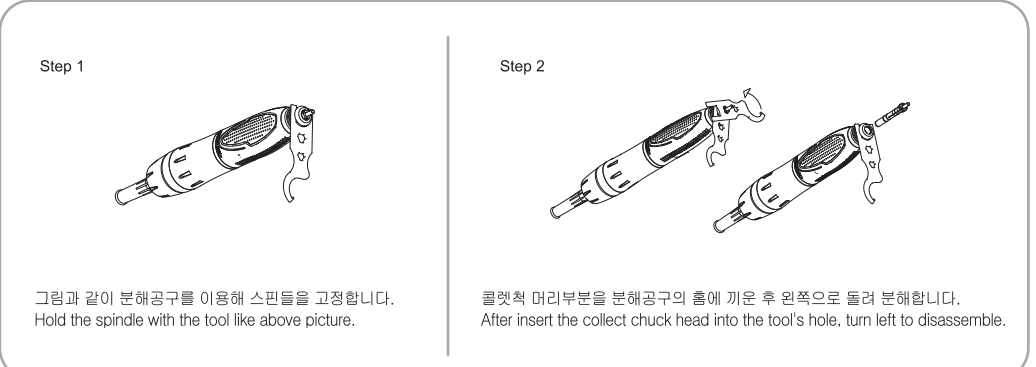
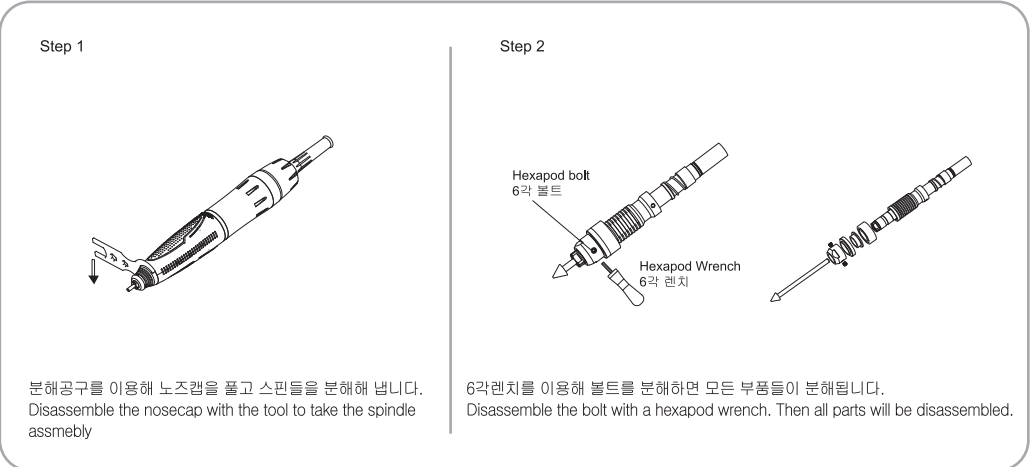
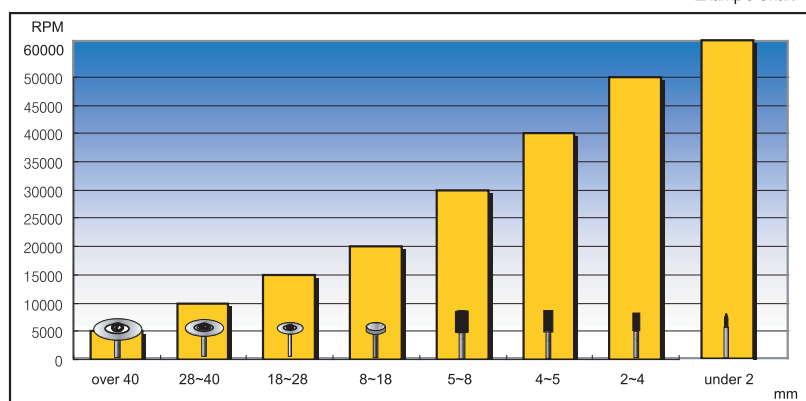
Problem	Check point	Solution
Power Lamp is not light when you turn on the power switch.	Is the power cord plug connected well?	Insert the power plug correctly.
	Isn't the fuse broken ?	Replace to new fuse.
	Trouble of power switch.	Please ask repair.
The motor does not work. (Please check the error code list.)	E1 It is motor's fault..	1.Check electrical connection of motor 2.If this error code is displayed continuously, Please ask repair.
	E2 It is overflow error.	Turn off the control unit, please keep it at cool place for 10 minutes and try again.
Foot pedal does not work.	Is the cord of foot pedal connected well ?	Connect the pedal plug into a correct jack.
	Is Foot control mode selected ?	Select foot pedal control mode. It is green lamp on pannel.
The handpiece makes noise.	Isn't foreign substance in the collect chuck?	Please remove it by asking repair to our agent in your area.
	Don't you use it at abnormal speed or Isn't the bur curved?	Replace the curved bur and use under permitted RPM for bur size.
	Check if the ball bearing is good condition.	Please ask repair.

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